


MULTI-PANEL CLIA WAIVED DRUG TEST CUP INSTRUCTIONS

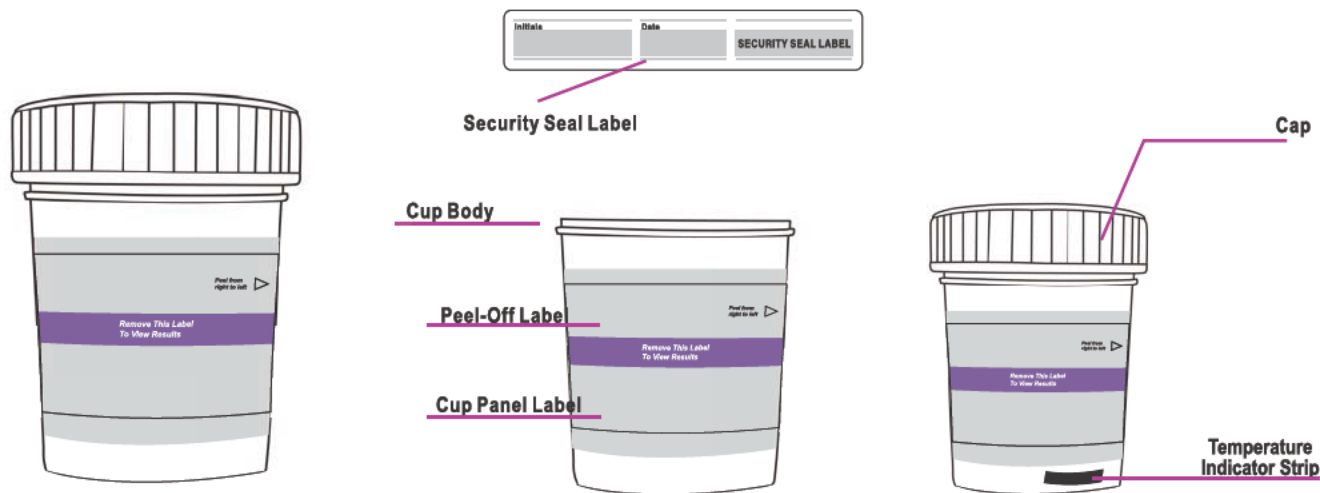


1 Donor provides urine specimen in cup. If desired, cap can be held by operator to initiate the test at a later time.



2 Once urine sample has been collected, screw cap firmly onto cup.




3 Check urine temperature at 2-4 minutes by reading green color on temperature indicator strip.

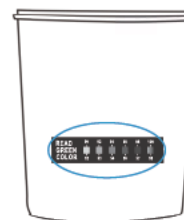

4 Donor dates and initials security seal. Operator checks cap for tightness and attaches security seal label over cap as shown (Optional).



5 Remove peel-off label.

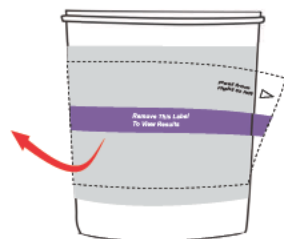

6 Read adulteration strip pads between 2 minutes, if applicable.



7 Read test results at 5 minutes. Refer to diagram below for result interpretation.



C	or	C	or	C	or
T		T		T	
NEGATIVE		POSITIVE		INVALID	



	Normal	Abnormal
COU	□	□
BG	□	□
pH	□	□
NI	□	□
GLUT	□	□
CEAA	□	□

8 Positive test results must be confirmed by another test method. Send cup and urine specimen with security seal label intact to a toxicology laboratory for confirmation.

